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Out of production, accessories available to end of stock

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Certification Mark: 

Scope of Certificate: Design and development, production and distribution of dental implants and abutments. Distribution of instruments for surgical dental implant

Applied Standard(s): EN ISO 13485:2012 + AC:2012
Medical devices - Quality management systems - Requirements for regulatory purposes (ISO 13485:2003 + Cor. 1:2009)
DIN EN ISO 13485:2012

The Certification Body of TÜV SÜD Product Service GmbH certifies that the company mentioned above has established and is maintaining a quality management system, which meets the requirements of the listed standard(s). See also notes overleaf.

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EC Certificate

Full Quality Assurance System

Directive 93/42/EEC on Medical Devices (MDD), Annex II excluding (4)
(Devices in Class IIa, IIb or III)

No. G1 14 03 84091 002

Manufacturer: Dental Planet
di Barozzini Marco & C. S.A.S.
Via Imola, 90/B
41125 Modena
ITALY

Facility(ies): Dental Planet di Barozzini Marco & C. S.A.S.
Via Imola, 90/B, 41125 Modena, ITALY

Product Category(ies): Dental implants and accessories
for dental implant
(abutment, healing and connection screws)

The Certification Body of TÜV SÜD Product Service GmbH declares that the aforementioned manufacturer has implemented a quality assurance system for design, manufacture and final inspection of the respective devices / device categories in accordance with MDD Annex II. This quality assurance system conforms to the requirements of this Directive and is subject to periodical surveillance. For marketing of class III devices an additional Annex II (4) certificate is mandatory. See also notes overleaf.

Report No.: ITA 241885

Valid from: 2014-04-01
Valid until: 2019-03-31

H.-H. Junker

Date, 2014-04-02
Hans-Heiner Junker

TÜV SÜD Product Service GmbH is Notified Body with identification no. 0123

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DENTAL PLANET Italian company leader in the field of oral implantology, specializing in the design of products such as dental implants, prosthetic accessories and compatible materials for bone regeneration. The experience gained over the years has allowed us to develop prosthetic - implant technologies, high-quality and innovative materials at affordable prices.

EDUCATION AND TRAINING

Courses and cultural evenings show a large range of prosthetic solutions in line with the philosophy of the company offering new technical results that give safe and valid alternative to the techniques already known and in use for many years.

The experience of dentists and dental technicians who work closely with us, together with the research activities of our department of technical planning and micro mechanics, have allowed us to achieve excellent results and to develop courses of interest to all operators in the sector.

RESEARCH AND DEVELOPMENT

Thanks to the suggestions of our customers and to solve their problems we are constantly improving our products and of our techniques.

Availing ourselves of supports and structures within institutions of higher learning, we are able to verify the validity of our products according to the latest technologies and techniques more associated. Moreover, the experience allows us to evaluate and design and innovate by producing materials that are designed and tested in our facilities and certificates for different countries.

PRODUCTION

Engineers, mechanical technicians, skilled technicians make up the staff.

To produce the components we employ bar lathes with numerical control of the last generation, high precision machines, with dynamic corrections of the tools, which allow the respect of tolerances of 0.001 mm (1 micron).

All of the innovations introduced in the early stages of production and the characteristics of the price are the result of continuous and in-depth studies as well as to significant and ongoing investments.

QUALITY AND CONTROL

Sophisticated management systems constantly monitor all stages of production, acting automatically on pending lawsuits, in case of deviation from the set parameters. The control of the production is based on the protocols defined SPC (Statistic Process Control) in which are fixed the amount and frequency of controls (depending on the type of product) and the criteria of acceptability. The dimensional controls are made on the three-dimensional measuring machines of the latest generation, capable of ensuring an accuracy of $\pm 0.0005\text{ mm}$ (0.5 microns). The quality of production and complies with EN ISO 13485:2012 / AC : 2012 and 93/42/EEC concerning medical devices.

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The WAREHOUSE

The storage of the majority of semi-finished products and finished products and entrusted to qualified personnel. All components are checked on entry, split and packaged in accordance with quality standards, all to allow operators to a specific preparation of orders. Orders for standard deliveries are made in Italy and abroad in 24 hours in 5 working days.

CUSTOMER SERVICE

Dental Planet works to meet all the needs of the customer and is present with its own distributors in Italy and abroad. Assistance pre and post-sale service is offered by qualified technical department and sales department are always available to answer any question and to provide information as detailed as possible for the customer to choose of the product, its application and use.

ORDERS: We accept orders by email and telephone.

EVADE THE ORDERS : The merchandise is usually available in stock. To the extent not available, delivery will be made as soon as possible and no later than 15/20 days. business communications unless otherwise specified.

DELIVERY TIME : 24/48 hours

PACKING: Freeware

SHIPPING: The merchandise travel at the risk and peril. Delivery is by express courier with relative charge on the invoice

WARRANTY: All merchandise is guaranteed. If not conform to the sample or description, can be returned within 10 days of receipt and we will refund or immediate replacement.

PRICES: Prices are subject to change without notice.

RETURNS: Returns are not accepted unless previously authorized. They will be given for the return and if accepted, the merchandise must be accompanied by ddt containing the number and date of the invoice relating to the goods at issue. We do not accept returned goods over 30 days. from the date of the delivery. The merchandise sent will always be shipped prepaid.

DISPUTES: All disputes are settled by the Court of Modena.

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Comparison between a novel biomimetic treatment and commercially exploited surfaces.

Chiesa R., Giavaresi G., Fini M., Sandrini E., Giordano C., Bianchi A., Giardino R., In vitro and in vivo performance of a novel surface treatment to enhance osseointegration of endosseous implants. Politecnico di Milano, Rizzoli orthopedic institute, and Vita-salute University. *Pathol oral radiol endod* 2007;103:745-56.

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PURPOSE

The purpose of this document is to demonstrate the high degree of biocompatibility of the treated titanium surface with the chemical method called TIROUGH; order we report the abstracts of articles published in journals national and international.

As for the in vitro analysis refer to the attached document "Surfaces Comparison: comparison between a novel and etched surface TiUnite-like surfaces".

It clarifies that the treatments and TiRough BioRough are actually the same treatment, the name differs only for marketing purposes.

REFERENCES :

Jordan C., E. Sandrini, Busini V., R. Church, Fumagalli G., Giavaresi G., M. Fini, Garden R., Cigada A., A new chemical etching process to improve endosseous implant osseointegration: in vitro on human evaluation osteoblast-like cells.

C. Giordano, Sandrini E., Del Curto B., E. Signorelli, Rondelli G., Di Silvio L., Titanium for osteointegration: Comparison between a novel biomimetic treatment and commercially surfaces exploited.

R. Church, Giavaresi G., M. Fini, Sandrini E., C. Giordano, A. Bianchi, R. Garden, In vitro and in vivo performance of a novel surface treatment to Enhance osseointegration of implants endosseous. Politecnico di Milano, Rizzoli orthopedic institute, and Life-Health University. *Pathol Oral radiol endod* 2007; 103: 745-56.

GENERAL INFORMATION :

The surface treatment TiRough is the surface modification of titanium and its alloys through a process that combines a blasting process and subsequent chemical etching with acids inorganic low concentration.

These techniques will not only ensure the formation on the surface of titanium and its alloys of a particular surface morphology, but also thanks to the strength of the chemical process guarantees decontamination absolute organic residues from the mechanical processing operations, if still present after the washing processes.

The surface of a component, whether dental implant, or orthopedic implant, treated with the method TiRough has a microporous surface, which stimulates osteoblast adhesion and differentiation (cells bone tissue) and stimulates the mineralization of bone tissue and therefore the maturing of this fabric.

THEORETICAL PRINCIPLES - STATE OF THE ART

The conditions of treatment with acids have been defined on the basis of the technical and scientific literature in matter and knowledge developed by this company in the course of its development activities of treatments chemical and electrochemical implantable medical devices.

The processing performed includes a first step of sandblasting with corundum white followed by a step of chemical etching with a mixture of acids, hydrochloric acid and sulfuric acid, at low concentration to a temperature of 50°C. The chemical process, besides giving a porous surface, removes residual abrasive used during the sandblasting phase.

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TiRough

Characterization of TiRough
Morphological analysis
Profilometry

With this technique, we analyzed the roughness profiles 1D, 2D and 3D maps of samples treated TiRough.

Figura 1: mappatura 2D e 3D / Figure 2 : 2D and 3D mapping

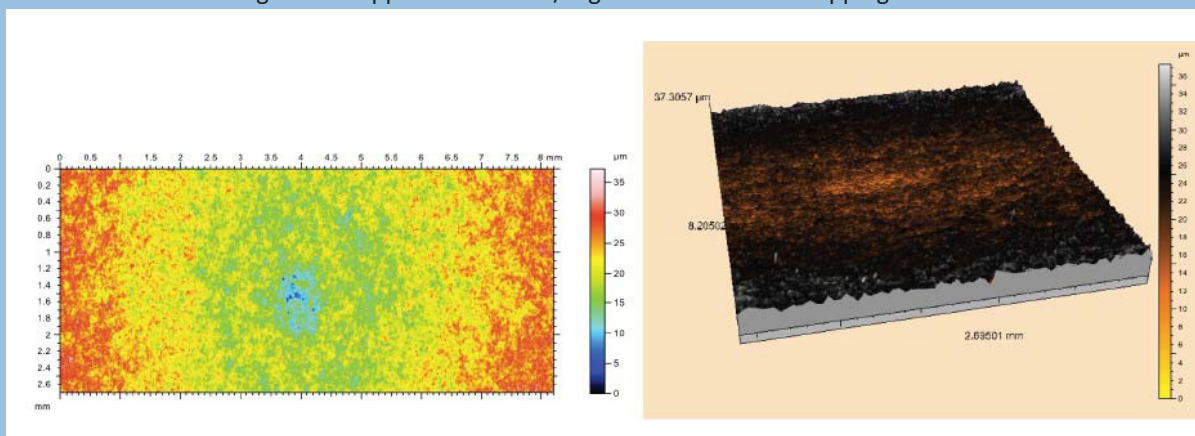
Figura 2: mappatura 2D e 3D / Figure 2 : 2D and 3D mapping

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
Scanning electron microscopy (SEM)

This section shows SEM images of the samples treated TiRough.

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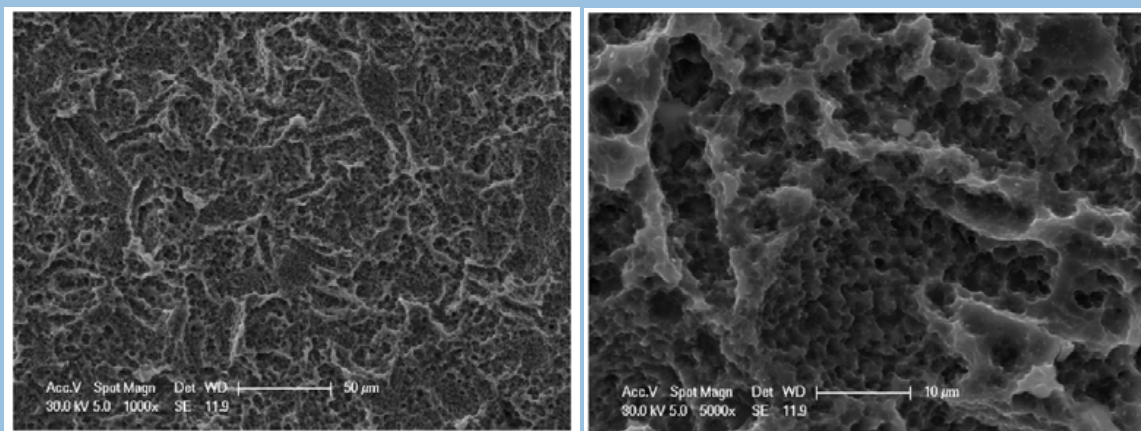
Compositional analysis of the surface

Energy Dispersive Spectroscopy (EDS)

The state of the surface compositional was evaluated by chemical analysis with EDS technique, this technique is used in the scientific literature to assess the degree of decontamination of metallic surfaces.

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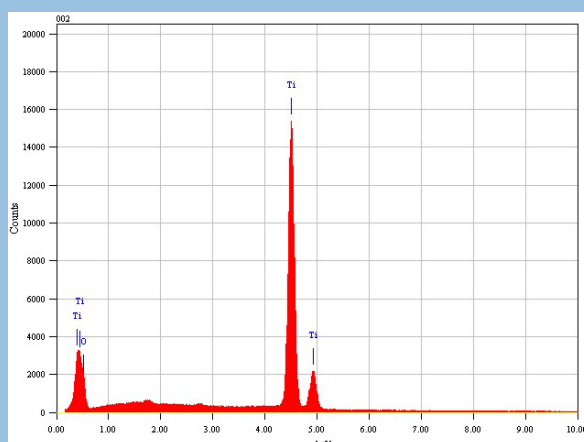Conclusions

In conclusion, the characterization analysis conducted allowed to verify:

- The process TiRough change the first layer of devices covered
- The roughness obtained is in line with what is considered, state of the art, suitable for promotion cell adhesion and bone healing.
- The process allows the complete decontamination TiRough residues blasting (corundum).
- The surface obtained is comparable as chemical and physical treatments already widely used commercially as SLA and Osseotite.

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GAMMA 25kGy

- Disposable medical devices in accordance with European Directive 93/42
- In vitro diagnostic medical devices in accordance with European Directive 98/79
- pharmaceutical raw material and medicinal products (human and veterinary use) according to the authorizations of Italian Drug Agency and Italian Healthcare Ministry.

Sterigenics applies and develops the gamma ray technology that ensures a safe and effective sterilization of a variety of industry products, at the same time improving and increasing both the characteristics and the applications of the products themselves.

Sterigenics is authorized by:

- Healthcare Ministry for the sterilization of active substances and ingredients for veterinary use (in conformity with GMP);
- AIFA (Italian Drug Agency) as Pharmaceutical for the sterilization of raw materials and medicinal products (in conformity with GMP)

Sterigenics is accredited by:

- Japanese Health, Labour and Welfare Ministry
- British HMRA, Medicines and Healthcare products Regulatory Agency
- in the US FDA, Food and Drug Administration (Registration and Device Listing Database).

Legislation

Medical and Diagnostics Sector:

Directive 93/42/CE "MEDICAL DEVICES" implemented by Legislative Decree 46 of 24 February 1997 amended by Legislative Decree 37 of 25 January 2010.

Directive 98/79/CE "IN VITRO MEDICAL/DIAGNOSTIC DEVICES" implemented by Legislative Decree 332 of 08 September 2000 amended by Legislative Decree 37 of 25 January 2010.

Standards

UNI EN ISO 9001:2008 - Quality Management System

UNI CEI EN ISO 13485:2012 - Medical devices – Quality Management Systems

UNI EN ISO 14001:2004 - Environment Management Systems

BS OHSAS 18001:2007 - Occupational health and safety Managements System

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All product changes in recent years have been based on customer feedback in order to make our implant systems more efficient and easier to use.

Pick simplified system:

The new packaging ensures easy access to plants and the cover screw.

The implants can be easily removed from the envelope thanks to the "no touch" conveyor, which allows you to bring the implant directly into the mouth without intermediate steps and start the call and then end with a ratchet or contra-angle.

The cover screw is situated in the upper part of the conveyor.

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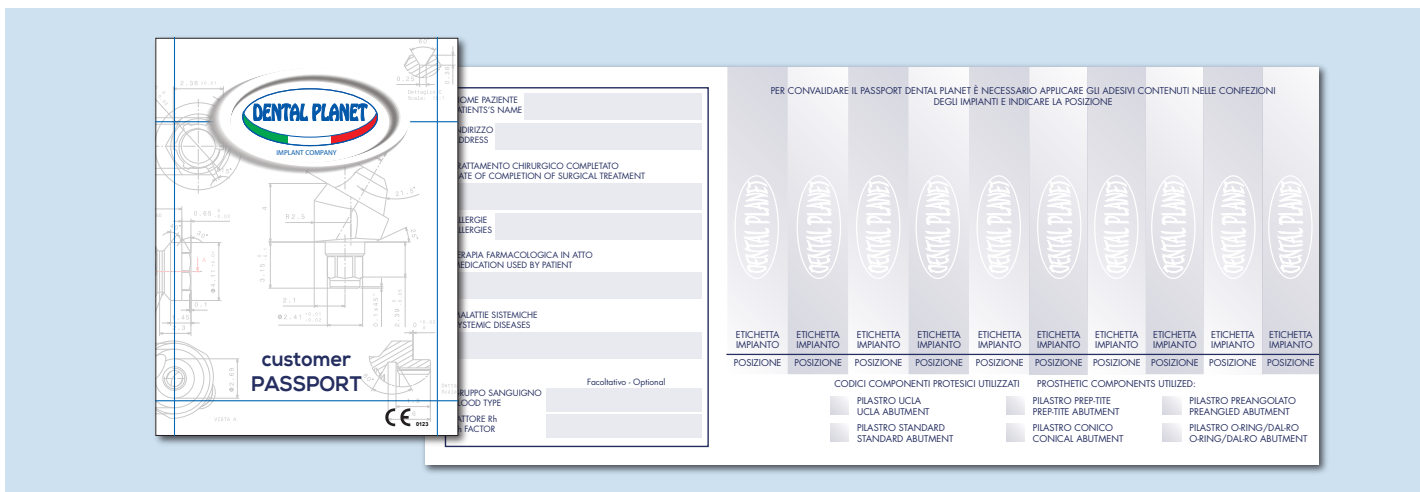
IMPLANT PASSPORT

Each implant is accompanied by an "implant passport" issued dall'implantologo to the patient after the surgery.

It is a guarantee of quality of equipment and components used, it also contains useful information for traceability and careful management.

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The label on the packaging of any medical device placed on the market show the symbols in accordance with EN ISO 13485:2003 / AC: 207 and 93/42/EEC concerning medical devices.

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COMPONENTS

Dental Planet manufactures and markets accessories for countless prosthetic implant platforms, all fully compatible. Unfortunately, being always continuously widening range accessories products we kindly ask you to contact us if you do not find in the catalog detail you're looking

Recall in addition that the Dental Planet is able to make small production, just send us the technical drawing or sample piece and as soon as the company will provide you with a quote and delivery time.

Healing Abutment :

Use the unveiling of the plant. The stump is screwed on the implant and left standing until the healing of the gums suitably adapted suture around the same. The use of these components and to rehabilitate soft tissue over the facility in order to insert the final prosthetic abutment.

Technical Specifications:

- Produced in grade 5 Titanium for perfect healing.
- The product is supplied decontaminated to reduce the microbial load
- Available in different heights to fit various gingival thicknesses
- The product is DISPOSABLE
- Supplied in single packaging NO STERILE, to sterilize in autoclave by the doctor before using it on the implant
- Store the product in a dry place and at room temperature, taking care to keep the packaging intact.

Titanium Abutment :

- Titanium Abutment fixed through the screw to the dental implant, allow the prosthesis anchoring

Technical Specifications:

- Made of Titanium Grade 5 to ensure excellent resistance to chewing load.
- The product is supplied decontaminated to reduce the microbial load
- Supplied in single pack with screw, NO STERILE. Sterilize the abutment screw and the final product in autoclave by the doctor before using
- The product is DISPOSABLE
- Store the product in a dry place and at room temperature, taking care to keep the packaging intact.

Titanium abutment they can be :

Final Abutment :

- Available in various forms and transmucosal emergencies to better adapt to the aesthetic needs of the patient

Temporary Abutment :

- Available rotating and non rotating, suitable for cementation of provisional cover

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COMPONENTS

Elements from melting Plexiglass :

Plexiglass castable are available rotating and no rotating, are preformed and must be modified in the final form by fusion . Intended for use by the dental.

Technical Specifications:

- Product Plexiglas, ideal material because no residue during melting.
- The product is supplied decontaminated to reduce the microbial load
- The product is DISPOSABLE
- Supplied in single pack with screw, NO STERILE. Sterilize the abutment screw and the final product in autoclave by the doctor before using it on the implant
- Store in a dry place at room temperature taking care to maintain package intact.

Elements from melting Plexiglass CrCo base :

Plexiglass castable CrCo base are available rotating and no rotating, are preformed and must be modified in the final form by fusion . Intended for use by the dental.

Technical Specifications:

- The product is supplied decontaminated to reduce the microbial load
- The product is DISPOSABLE
- Supplied in single pack NO STERILE, sterilize the short abutment screw and the final product in autoclave by the doctor before using
- Store in a dry place at room temperature taking care to maintain package intact.

Appearance	Solid
Color	Grey metallic
Physical and mechanical properties	
Odor	Odorless
Smell	
Melting range	1.310-1.330°C
Casting temperature	1.450°C
Density	8.8 g/cm ³
Tensile strength (mpa)	700 mpa
Yield strength rp 0.2%	580 mpa
Modulus of elasticity	210 mpa
Percentage elongation A%	8%
Reduction of area%	10%
Vickers hardness Hv 30	300
Elemental composition by weight	
Chrome (Cr)	22%
Molybdenum (mo)	6.5%
Tungsten (W)	6%
Cobalt (Co)	63%

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COMPONENTS

Prothetical screws:

used to secure the prosthesis implant in final form

Technical Specifications:

- Produced in Grade 5 titanium to ensure excellent resistance to chewing load.
- The product is supplied decontaminated to destroy the microbiological burden
- Available in two lengths, short for the final prosthesis, long for the stumps impression.
- Included in the package of the prosthetic abutments, impression and elements from casting plexiglas and gold.
- Also supplied in multiple packs NO STERILE, to sterilize in autoclave by the doctor before using
- For screwing instructions, refer to the description in the "Prosthetic Screws" section of the product catalog
- The product is DISPOSABLE
- Store in a dry place at room temperature taking care to maintain package intact.

Laboratory analog :

Used as a replication of the dental implant during prosthetic construction

Technical Specifications:

- Produced in Steel
- The product is supplied decontaminated to destroy the microbiological burden
- Supplied in single pack NO STERILE
- The product is DISPOSABLE

Imprint transfert :

Used to detect the imprint for the realization of the dental prosthesis


Technical Specifications:

- Produced in Steel
- The product is supplied decontaminated to destroy the microbiological burden
- The product supplied with long screw
- Supplied in single pack NO STERILE
- The product is DISPOSABLE

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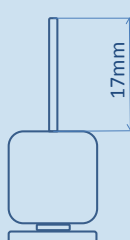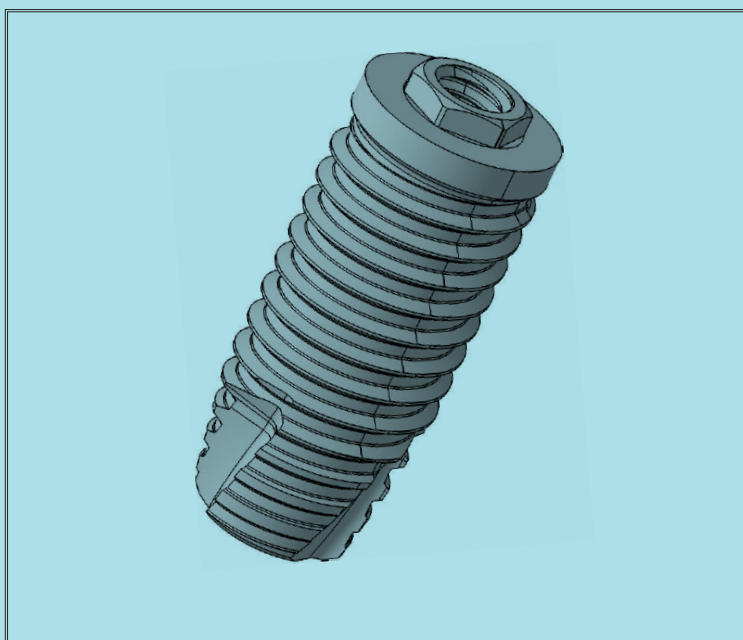
PROPERTY

CYLINDRICAL FORM
ROUGH SURFACE
WORKED NECK
WIDE LOOP

- Property autocondensante and high primary stability.
- Fast and easy insertion thanks to the 2-loop principles.
- Better control during implant insertion.
- Suitable for all surgical procedures
- Excellent in all types of bone

Each package consists of:

- An aluminum envelope containing the implant with titanium mounter used as a transfer or final abutment.
- A screw cap
- A leaflet
- Patient label to apply to the implant passport

	Compatibile 3i® e Branemark® 3i® and Branemark® Compatible			Compatibile 3i® Compatible 3i®
Lunghezza Lenght	Ø 3,30 codice/code	Ø 3,75 codice/code	Ø 4,0 codice/code	Ø 5,0 codice/code
7	---	---	ONE4007	---
8	---	ONE3708	ONE4008	ONE5008
10	ONE3310	ONE3710	ONE4010	ONE5010
11,5	ONE3311	ONE3711	ONE4011	ONE5011
13	ONE3313	ONE3713	ONE4013	ONE5013
15	---	ONE3715	---	---

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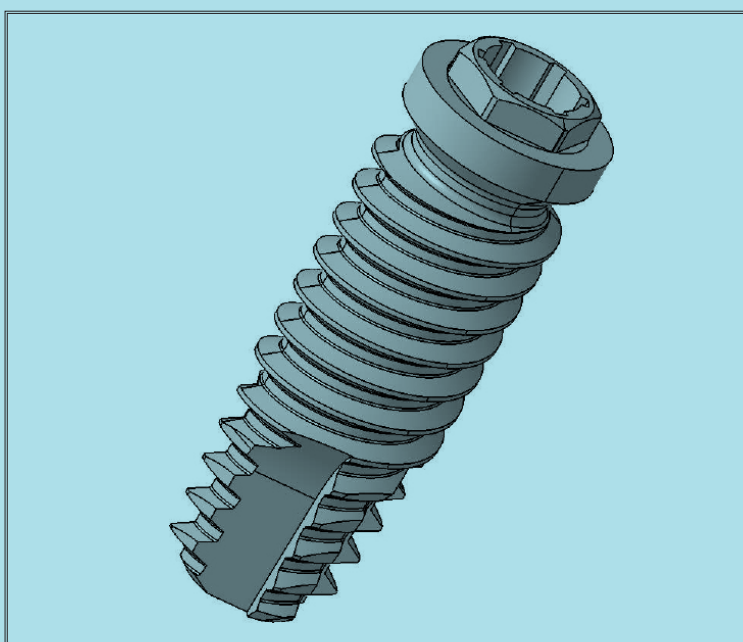
Compatible prosthetic accessories see page 47-71

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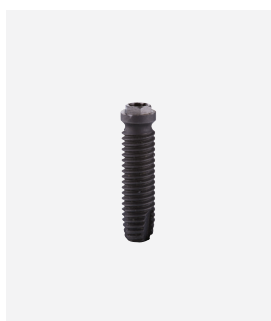
PROPERTY

CYLINDRICAL FORM
ROUGH SURFACE
WORKED NECK
WIDE LOOP

- Property autocondensante and high primary stability.
- Fast and easy insertion thanks to the 2-loop principles.
- Better control during implant insertion.
- Suitable for all surgical procedures
- Excellent in all types of bone

Each package consists of:

- An aluminum envelope containing the implant
- A screw cap
- A leaflet
- Patient label to apply to the implant passport


	Compatibile Branemark®/ Branemark® compatible		
Lunghezza Lenght	Ø 3,30 codice/code	Ø 3,75 codice/code	Ø 4,0 codice/code
8	---	---	SG4008
10	SG3310	SG3710	SG4010
11,5	SG3311	SG3711	SG4011
13	---	SG3713	SG4013
15	---	SG3715	SG4015
18	---	SG3718	SG4018

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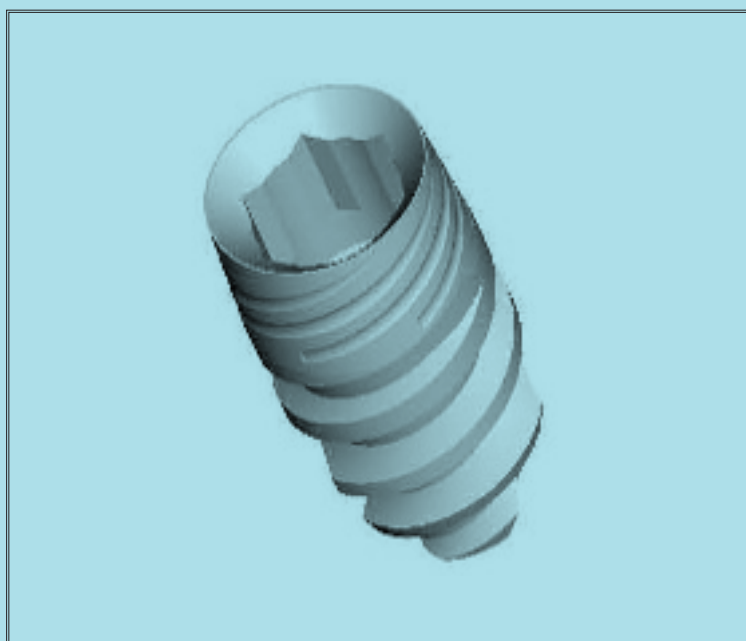PROPERTY

CILYNDRICAL FORM
ROUGH SURFACE
WORKED NECK
WIDE LOOP

- Property autocondensante and high primary stability.
- Fast and easy insertion thanks to the 2-loop principles.
- Better control during implant insertion.
- Suitable for all surgical procedures
- Excellent in all types of bone

Each package consists of:

- An aluminum envelope containing the implant with titanium mounter used as a transfer or final abutment.
- A screw cap
- A leaflet
- Patient label to apply to the implant passport

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
Compatible prosthetic accessories see page 86

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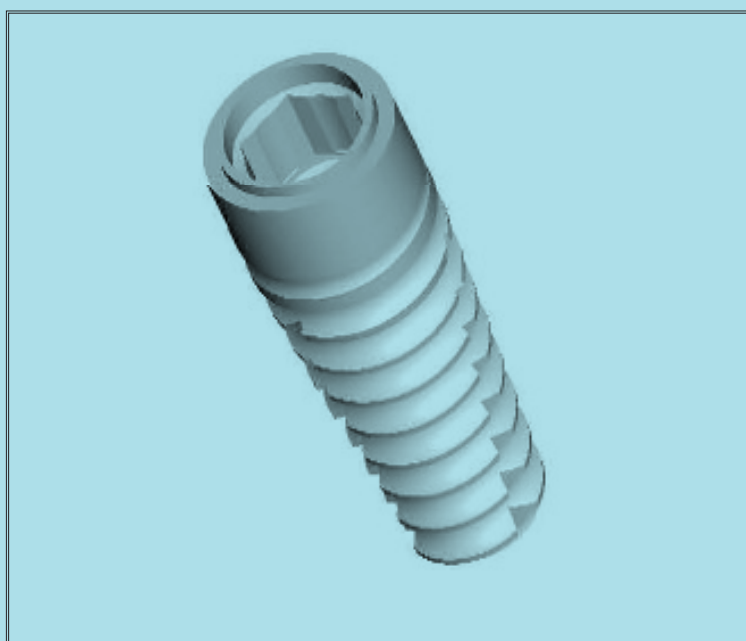PROPERTY

CONICALFORM
ROUGH SURFACE
WORKED NECK
WIDE LOOP

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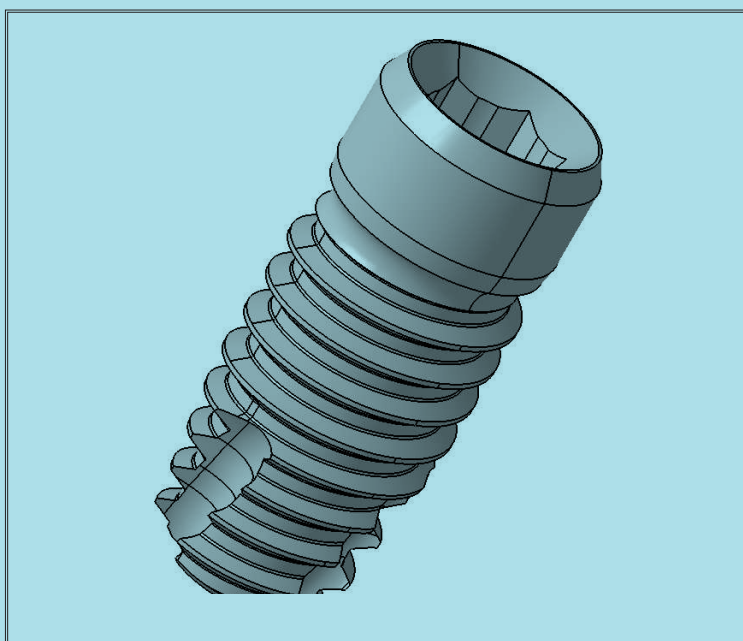
PROPERTY

CYLINDRICAL FORM
ROUGH SURFACE
WORKED NECK
MEDIUM LOOP

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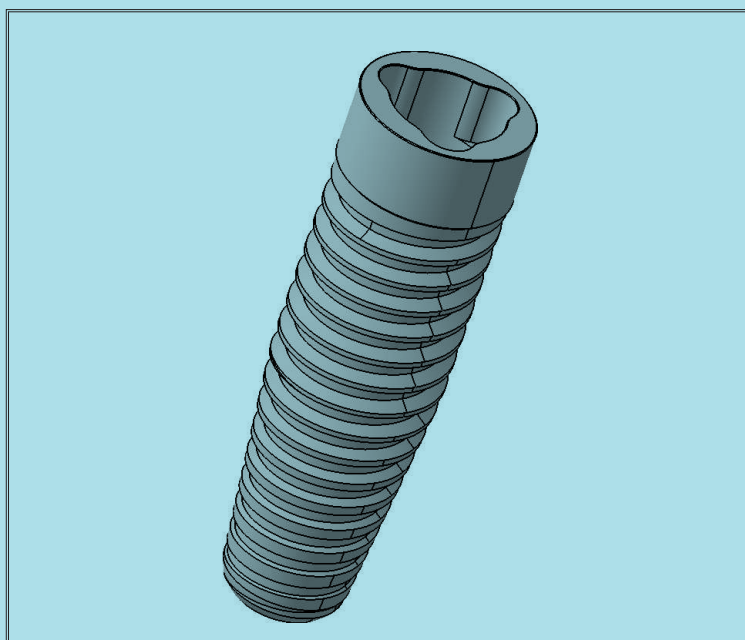
PROPERTY

CYLINDRICAL FORM
ROUGH SURFACE
WORKED NECK
MEDIUM LOOP

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Each package consists of:

- An aluminum envelope containing the implant with titanium moulder used as a transfer or final abutment.
- A screw cap
- A leaflet
- Patient label to apply to the implant passport

	Compatibile Nobel Replace® / Nobel Replace® compatible		
Lunghezza Lenght	ø 3,50 codice/code	ø 4,30 codice/code	ø 5,0 codice/code
8	EVOC3508	---	EVOC5008
10	EVOC3510	EVOC4310	---
11,5	EVOC3511	EVOC4311	---
13	---	EVOC4313	---
15	EVOC3515	---	---

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NOVOCOR PLUS

NOVOCOR PLUS patented by B&B Dental is a natural grafting material (100% mineral), made from natural coral granules useful for the repair of bone defects in the orthopedic sector. In particular in odontostomatology it is used as a filler for implant sites after the placement of dental implants

CHARACTERISTICS

- Natural coral grafting material (100% mineral).
- Slow resorption and direct integration of new bone.
- Long-term dimensional stability.
- Osteoconductive.
- No foreign body or inflammatory reaction.
- Hydrophilic surface, optimal cell adhesion and blood absorption.
- Safe and sterile.
- Easy handling.

CLINICAL APPLICATIONS

- Sinus lift.
- Horizontal augmentation.
- Intraosseous defects.
- Peri-implant defects.
- Extraction sockets.
- Vertical augmentation.
- Furcation defects.

PACKAGING

Each box contains 4 capsules from 0.5 g.
Grain size: 0.2 – 0.5 mm.

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T-BARRIER COLLAGENE MEMBRANES

T-BARRIER collagen membranes are made from type-1 native heterologous equine collagen, which ensures perfect biocompatibility and complete resorption. They are indicated for guided tissue regeneration procedures and are applied to enhance the healing of operation sites .

CHARACTERISTICS

- Perfect biocompatibility.
- Complete resorption after 4/6 months.
- Osteoconductive and osteoinductive activity.
- Anti-inflammatory, eutrophic and cicatrizing properties.
- Easy to apply on the defect area.

CLINICAL APPLICATIONS

Oral surgery: containment action (tent effect) for heterologous and autologous grafts.
Paradontology: treatment of gum recessions.
Implantology: protection of the sinus membrane prior to insertion of graft material.

PACKAGING

Each box contains two membranes.
Dimension: 23×23 mm.
Thickness: 0.25 mm.

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T-BARRIER TITANIUM MEMBRANES

Developed by B&B Dental T-Barrier Titanium are membranes Grade 2 commercially pure titanium, non-absorbable and micro-perforated and they represent one of the most effective solutions for guided bone regeneration, especially for the stabilization and protection of graft material in a bone defect .

CHARACTERISTICS

- Perfect biocompatibility.
- Easy to apply on the defect area.
- Easy to cut.

CLINICAL APPLICATIONS

The grid is indicated in all cases of guided bone regeneration, especially in view of a subsequent rehabilitation with intraosseous implants. It can be used in both upper and lower dental arch of patients with bone defects that limit or exclude the possibility of implantology treatments.

PACKAGING

They are supplied in blister packs, non-toxic, non-pyrogenic and sterile. Each pack contains two membranes. Dimensions: 24x29 mm. Thickness: 0,13 mm.

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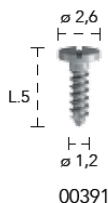
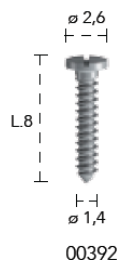
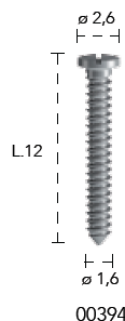
00396

OSTEOSYNTHESIS SCREW

Osteosynthesis screws feature a highly-penetrating thread allowing for easy screw insertion and a cross-slotted head for holding the screw in engagement with the screwdriver shaft.

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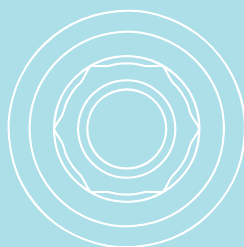
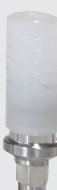
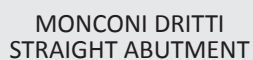
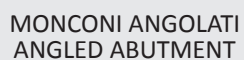
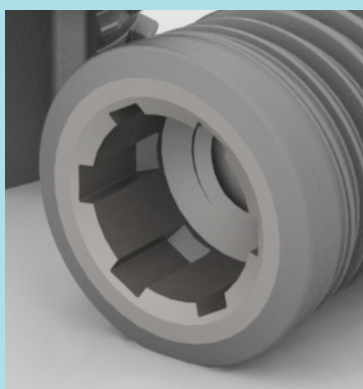
ANALOGO / ANALOGUE

DP-79/B

MONCONE FRESABILE
MILLABLE ABUTMENT

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